

## INTENDED USE

This blood collecting device is a single-use blood lancing device intended for obtaining microliter capillary whole blood samples.

## DEVICE DESCRIPTION

This device is a sterile and disposable blood-lancing device. When collecting a sample, the device is held onto the skin by a mild adhesive. A sterile lancet punctures the skin with a press of the button, simultaneously this creates a slight vacuum that helps capillary blood flow under gravity into a compatible tube. After a predefined time period, or when the tube is full, the device is peeled off the skin. The tube is removed and capped for transport. The lancet is single use and automatically retracts into the device in a safe position after use.

## DISPOSAL

Follow local guidelines for discarding used sharps as a biohazard material.

**MAX LENGTH: 5 MM**

**MAX DEPTH : 2 MM**

## MATERIALS PROVIDED

- BLOOD COLLECTION DEVICE
- BIOHAZARD BAG
- REHEATABLE HEAT PACK
- COLLECTION TUBE
- BANDAGE
- ALCOHOL SWAB



**Tasso, Inc.**  
400 2nd Ave W., Seattle, WA 98119 USA  
+1 800-257-2370 | [www.tassoinc.com](http://www.tassoinc.com)

20297



## WARNINGS

Not intended for more than one use. Do not use on more than one patient. Improper use of blood lancets can increase the risk of inadvertent transmission of bloodborne pathogens, particularly in settings where multiple patients are tested.

## SYMBOLS

	CATALOG NUMBER		SINGLE USE ONLY
	LOT NUMBER		DO NOT RESTERILIZE
	DO NOT USE IF PACKAGE IS DAMAGED		STORE AT 18-28°C
	USE BY DATE		TRANSPORT AT 25-85% RH
	CONSULT INSTRUCTIONS FOR USE		PRESCRIPTION USE ONLY
	THIS DEVICE CONTAINS SHARPS		STERILIZED IRRADIATION


## PRECAUTIONS

- For one-time use only. Discard entire device after use.
- Do not use after expiration date.
- Keep out of reach of children.
- Minor bruising or residual marks may occur at the sample collection site.
- Do not resterilize.
- For external use only.
- Use while seated as fainting may occur with any blood sampling procedure.
- Not for use on infants.
- Prescription use only.
- Refer to manufacturer's instructions for individual components.
- Do not use if device packaging has been opened or damaged.
- Users must validate use of product for their specific diagnostic assay.
- Use personal protective equipment when handling biohazardous materials.



# Start here to begin your journey to cognitive health.

Watch our instructional video: [www.neurogenbiomarking.com/kit](http://www.neurogenbiomarking.com/kit) or scan the QR code to access.

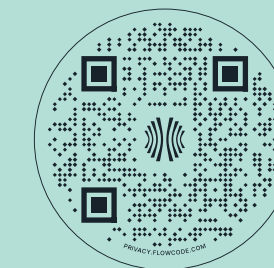

STEP-BY-STEP GUIDE FOR USING YOUR AT-HOME BLOOD SAMPLING KIT BY **tasso**


## IMPORTANT NOTICE BEFORE YOU USE

Please collect your sample only on **weekdays**. After collection, drop it off at your nearest FedEx location on the **same day before 2:00 PM local time** to ensure timely processing.

### 1. PREPARE




01 Wash hands before use.

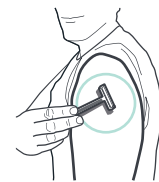


02 Device will be placed on upper arm — for best results remove any excess hair.



03 Twist off cap from tube.

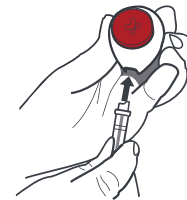


04 Press tube into device until snug with fill lines facing out.

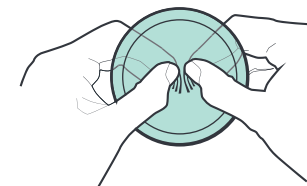

05 Activate the heat pack by clicking the metal disc. If provided warmer is firm, see back of warmer for instructions to reactivate.



### INCREASE BLOOD FLOW AT COLLECTION SITE

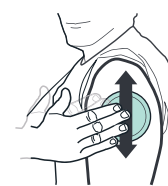

06 Rub heat pack up and down upper arm for 3 minutes to warm and increase blood flow.




07 Clean area with alcohol pad and allow to dry.

### 2. COLLECT




08 Remove clear cover from red button.

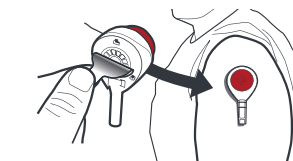


09 Peel tab away from device and stick device to upper arm. Hang arm straight down at side.




**YOU MAY NOT FEEL THE DEVICE. LEAVE ON 5 MINUTES OR LESS IF FULL.**

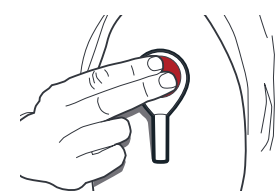

10 Press button all the way down **ONCE** and release. Set timer for 5 minutes. Blood may not appear for 1-2 min.

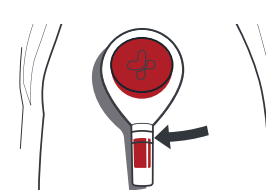


11 Watch tube filling using mirror as needed. Remove device at 5 total minutes or sooner if blood reaches top line before 5 minutes.

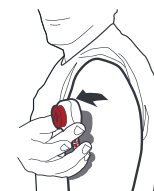


12 Remove device by peeling from one side. If needed, stop bleeding with bandage.



13 Remove tube from device with a slight twist and pull down.

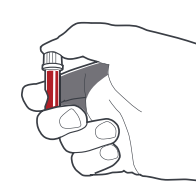


14 Snap cap fully onto tube.

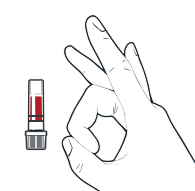

15 Turn the tube upside down. Flick tube **5** times.




16 Turn tube upside down **10** times.

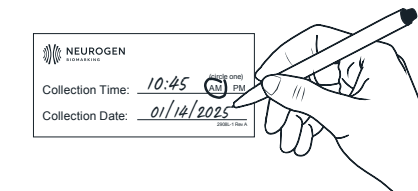

17 Write down the collection time and date on the label and place it in the plastic bag together with the blood sample.

### 3. TRANSPORT

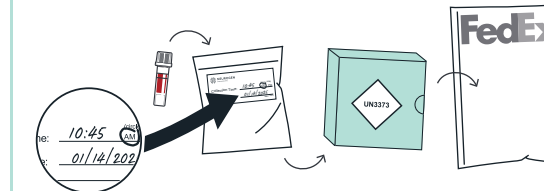

18 Follow the instructions on the back of the green box to prepare your blood tube, plastic bag, green box, FedEx mailer for shipping.



**HEAT PACK GETS VERY HOT. TO AVOID BURNING, TAKE IT ON AND OFF THE ARM INTERMITTENTLY.**